



## Thermostrip® Moving Line Disposable Thermometer

### Instructions for Use

#### How to Read

1. Remove backing liner. Apply to center of forehead, just above the eyebrows and read temperature when yellow line stops moving, generally 15 seconds. **Read the indicator while it is still on the forehead.**
2. Center of yellow line indicates the correct temperature

#### Intended Use

- Measure human forehead surface temperature and temperature trends

#### Precautions

- Environmental temperatures might affect the performance of this product
- After use, discard

Thermostrip® thermometers are to be used on the forehead area only and estimate the core body temperature by measuring the temperature of the forehead. It is an ideal way to check for changes in temperature. A Thermostrip® Forehead Thermometer is not as accurate for determining the actual core body temperature as a clinical (oral / rectal) thermometer. Instead it is designed to be an unbreakable, quick reading device for temperature screening between uses of a clinical thermometer.

Forehead skin temperatures are lower than oral temperatures.

Accurate to  $\pm 1^{\circ}\text{F}$  /  $\pm 0.6^{\circ}\text{C}$ , the readings on Thermostrip® thermometers have been adjusted to approximate the difference between forehead and oral temperatures during normal to feverish conditions. Thermostrip® products indicate an approximate oral temperature. If you think that your oral temperature is different from that shown by Thermostrip® you should check by using a clinical thermometer. If fever is present, consult your physician for appropriate treatment.

**For External use only**

**This product contains no latex**

#### Storage / Handling:

Products should be stored out of direct sunlight, dampness, or heat.

#### Cleaning:

Wipe clean using soap and water or rubbing alcohol. Do not soak.

Manufactured by, LCR Hallcrest, Glenview IL 60026